



Vitrex® Steriheal Baby Sterile Cutting Safety lancet

For high volume blood flow with minimum depth of penetration

ACTIVATION BUTTON: Secured by safety tab.

NO CHOCKING HAZARD: Non detachable safety tab.

USER SAFETY: Blade is fully shielded before and after use.

EASIER HANDLING: Anti-slip texture and one hand operation

USER CONVENIENCE: Pre-loaded, button activated design.

PATIENT SAFETY: Single-use only. Sterilised by gamma irradiation.

PATIENT COMFORT: Ultra sharp blade with high speed cutting securing minimal trauma and fast healing.

NARROWED LANCING SLOT: Easier positioning

APPLICATION: Primarily newborns and preemies. Can also be recommended for other applications where short penetration depth and/or minimal compression of tissue is important.



Pat Coakley Medical Ltd.

Units E3 E4 Network Enterprise Park
Kilcoole A63 H244 Co Wicklow
IRELAND
+353 12018070
sales@patcoakleymedical.ie



Recommended use

1. Release the trigger lock mechanism by pushing down the safety tab.
2. Reference position for infant heel blood sampling.
3. Raise the baby's foot to a suitable position, place the blade slot flush against the heel site and activate the lancet.
4. Collect the blood sample with a capillary tube.
5. Dispose of the safety lancet in a suitable safety container.

WARNING:

Do not use if safety tab has previously been activated or if the shield blister packing has been damaged.



Technical Specs.

Colour Code	Specification	Article No.	Application
Green	0.65 mm Depth / 1.40 mm Incision Length	335213	Micro-Premie
Orange	0.85 mm Depth / 1.75 mm Incision Length	334213	Premie
Yellow	1.00 mm Depth / 2.50 mm Incision Length	333213	Newborn
Pink	1.14 mm Depth / 2.80 mm Incision Length	332213	Newborn Plus
Blue	2.00 mm Depth / 3.00 mm Incision Length	331213	Toddler

Shelf life 5 years

Pat Coakley Medical Ltd.

Units E3 E4 Network Enterprise Park
 Kilcoole A63 H244 Co Wicklow
 IRELAND
+353 12018070
 sales@patcoakleymedical.ie