

NeoSeal[®]

Nasal Prong Seal



FEATURES AND BENEFITS

- Designed to improve the seal for more effective respiratory support
- Protects the skin of the septum and nares
- Compatible with most nasal prongs
- Not made with natural rubber latex or plasticizer DEHP

Cat No.	Size	Qty/Unit
N420	Micro	150/box
N421	Small	150/box
N422	Large	150/box



Made in USA

Patent Pending

NeoSeal[®]

Nasal Prong Seal

INDICATIONS FOR USE

The NeoSeal is intended to create a seal around the nares and protect the skin of the septum and nares. It is intended for use on pediatric (neonates, infants and children) patients.

DIRECTIONS FOR USE

Step 1

Peel NeoSeal from liner.

Step 2

Slip over nasal prongs with adhesive facing cannula.

Note: Soft, smooth side without adhesive should contact patient's skin.

Step 3

Place cannula per hospital protocol.

Step 4

Replace NeoSeal daily or per hospital protocol, whichever is sooner.

Note: Replace immediately if soiled or wet.



Pat Coakley Medical Ltd

Units E3 E4 Network Enterprise Park
Kilcoole A63 H244 Co Wicklow
sales@patcoakleymedical.ie

Tel: +353 1 2018070
www.patcoakleymedical.ie

GENERAL INFORMATION

Product Materials

PVC foam, PSA adhesive (does not contact skin)
Made with natural rubber latex: No
Made with plasticizer DEHP: No
Consult MSDS for complete material information.

Size

N420 Micro: 20 mm x 10 mm
Prong opening diameter: 4.5 mm
Septal space: 4 mm

N421 Small: 25 mm x 12.5 mm
Prong opening diameter: 5.5 mm
Septal space: 5 mm

N422 Large: 30.5 mm x 12.5 mm
Prong opening diameter: 6.5 mm
Septal space: 6 mm

Packaging

Dispenser box: Yes
Perforated liner

Visit neotechproducts.com for the most up-to-date information or to download the Material Safety Data Sheet (MSDS).

Boxes are printed with environmentally friendly soy ink on recyclable material.

CAUTIONS & WARNINGS

Federal law restricts this device to sale by or on the order of a physician.

Replace immediately if soiled or wet.

Discontinue immediately if skin irritation occurs.

Single patient use only.

