


NeoShades[®] with Headstrap

Phototherapy Eye Shields

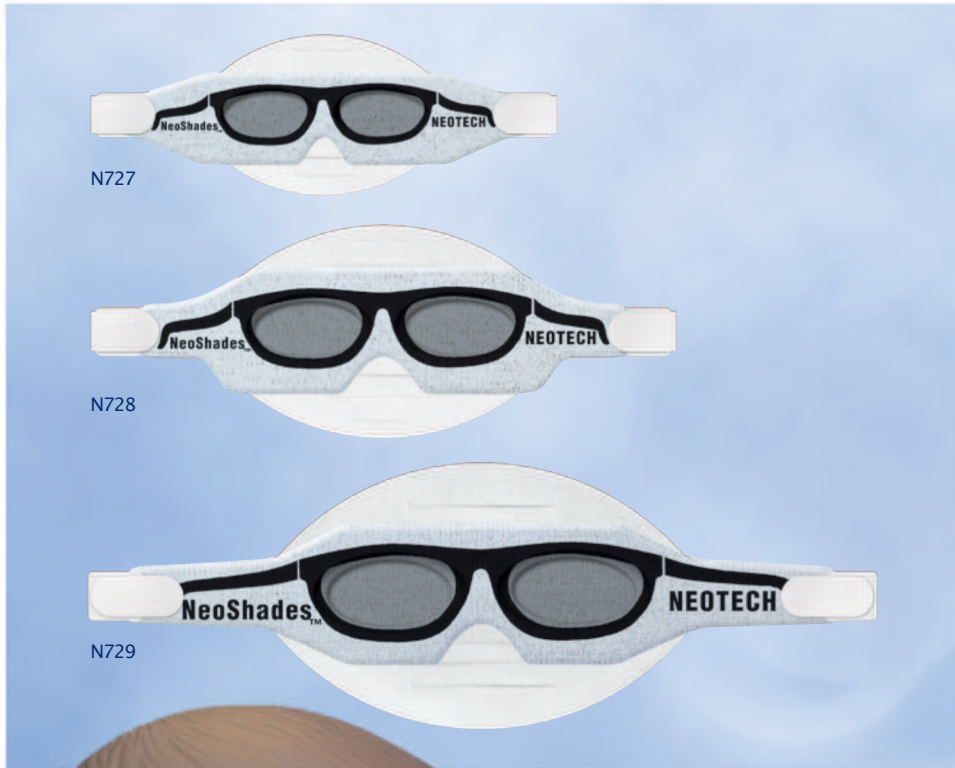

FEATURES AND BENEFITS

- Soft, padded eye shield material with cute sunglasses design
- Ocular pockets designed for patient comfort and safety
- Soft, patented NeoFoam[®] headstrap holds NeoShades in place
- Adjustable headstrap engineered to minimize slipping and help reduce head molding
- Unique headstrap shape designed to ensure ideal fit
- Not made with natural rubber latex or plasticizer DEHP
- Individually packaged

Cat No.	Item	Qty/Unit
N727	Micro	50/box
N728	Small	50/box
N729	Large	50/box



U.S. Patent #8,070,288;
 Additional Patents Pending

NeoShades® with Headstrap

Phototherapy Eye Shields

INDICATIONS FOR USE

The NeoShades are indicated for use as eye protection during phototherapy sessions. They are intended for use on pediatric (neonates, infants and children) patients.

DIRECTIONS FOR USE

Step 1

Select proper size.

Step 2

Place headstrap behind head with smooth side of foam against head.

Step 3

Cover eyes with NeoShades.

Step 4

Gently attach hook fabric on end of headstrap to NeoShades.

Repeat for opposite side.

Note: Straps can be trimmed for proper fit.



Pat Coakley Medical Ltd

Units E3 E4 Network Enterprise Park
Kilcoole A63 H244 Co Wicklow
sales@patcoakleymedical.ie

Tel: +353 1 2018070
www.patcoakleymedical.ie

GENERAL INFORMATION

Light Blocking

99.9%

Product Materials

NeoShades: Nylon, polyurethane
Headstrap: NeoFoam, hook fabric
Made with natural rubber latex: No
Made with plasticizer DEHP: No

Size

N727 Micro: 108 mm
(fits head circumference < 26 cm)
N728 Small: 127 mm
(fits head circumference 26–34cm)
N729 Large: 178 mm
(fits head circumference > 34 cm)

Packaging

Individually packaged: Yes
Package type: Polyethylene bag
Dispenser box: Yes

Visit neotechproducts.com for the most up-to-date information.

Polyethylene bags are biodegradable. Boxes are printed with environmentally friendly soy ink on recyclable material.

CAUTIONS & WARNINGS

The NeoShades are not intended for use on adults.

Federal law restricts this device to sale by or on the order of a physician.

NeoShades should only be used while the patient is under the continuous, direct supervision of healthcare professionals.

Single patient use only.



EMERGO EUROPE
Prinsessegracht 20
2514 AP, The Hague
The Netherlands



PPE Regulation (EU) 2016/425
Notified Body: BSI Group (2797),
Say Building, John M. Keynesplein 9
1066 EP Amsterdam, Netherlands

Neotech Products LLC • 28430 Witherspoon Pkwy • Valencia, CA 91355, USA