



Vacuderm – How To Use

1

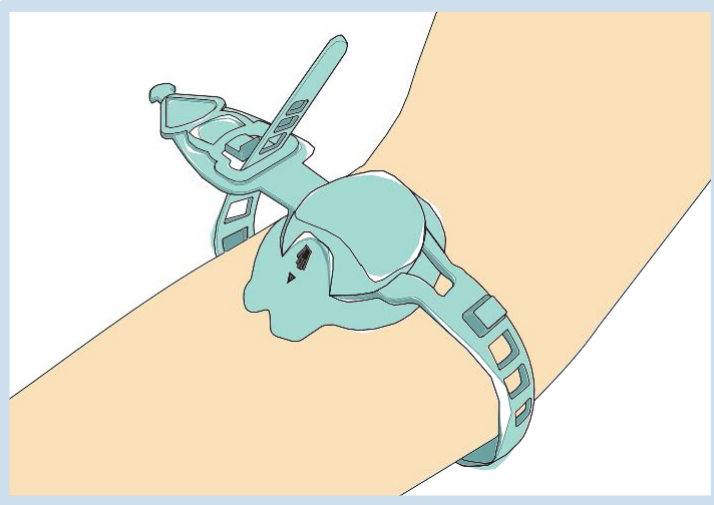


Ensure the hand/arrow V section is **pointing towards the patient's hand** to allow for accurate alignment to the vein, and effective pumping/filling of the vein.

2

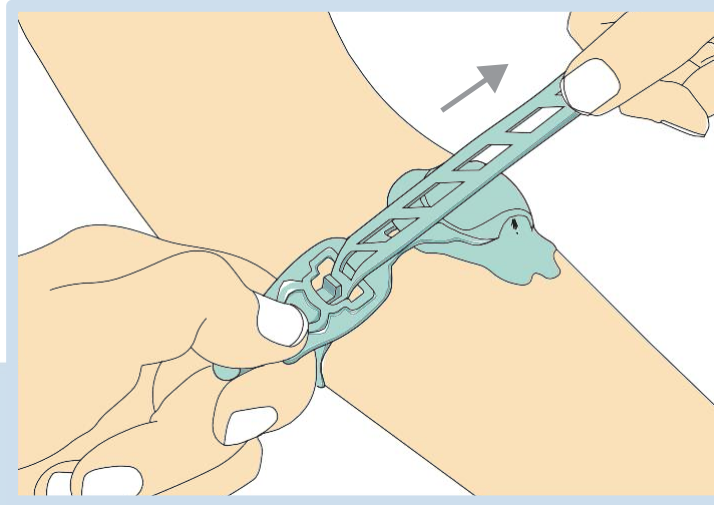


Using two hands, thread the strap through the buckle and stretch, ensuring the tourniquet is pulled as tightly as it will allow, to give the required tourniquet result.

3

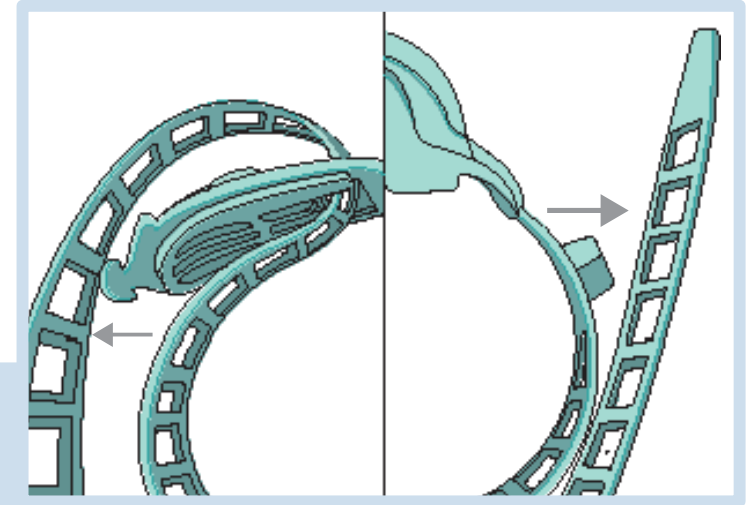


The strap should be hooked back onto itself to give a more secure placement and additional tightening.

4

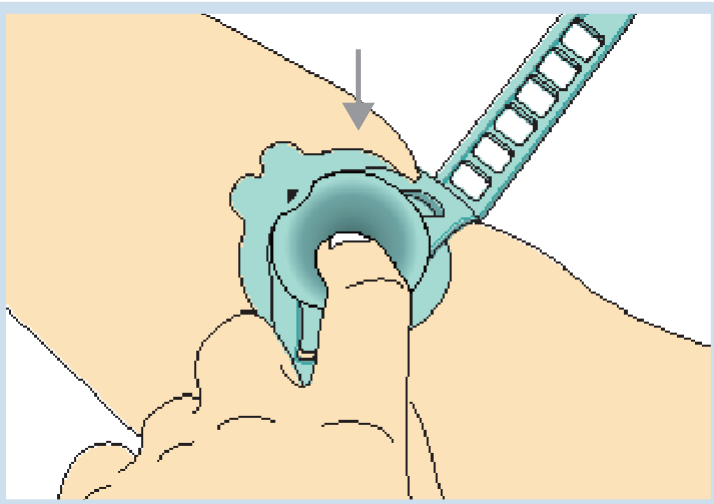


Pump the dome at least 10 to 15 times, gently and slowly towards the back of the dome. Continue pumping a few more times if the desired effect has not yet been reached. The vein should feel bouncy, ready for cannulation.

5

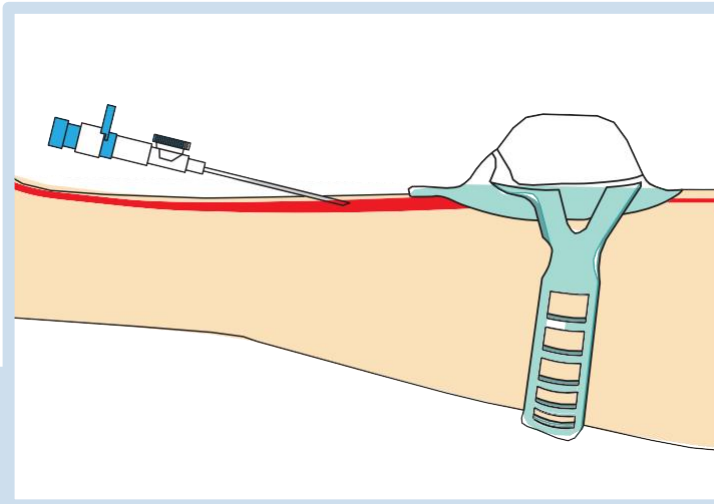

Insert the cannula as per standard practice.

6

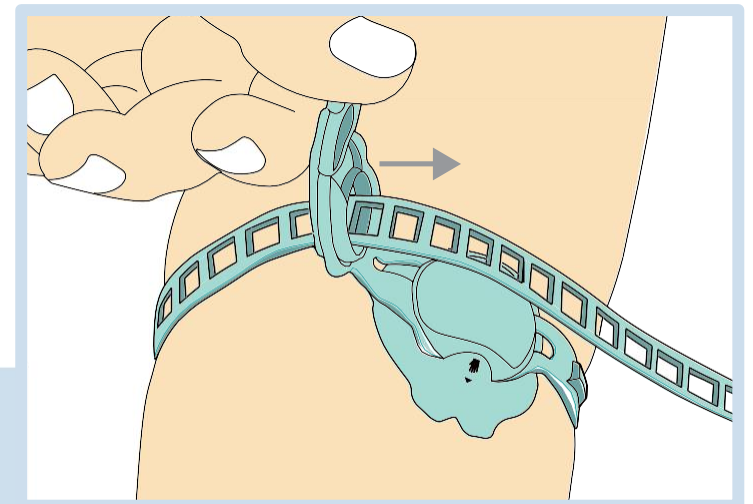

Release is single-handed by lifting the buckle. To complete the cannulation procedure, use a dressing to protect the IV insertion site as per standard practice.

For a video presentation please visit <https://www.youtube.com/watch?v=yX9kzxc73R8>

www.olberon.com